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

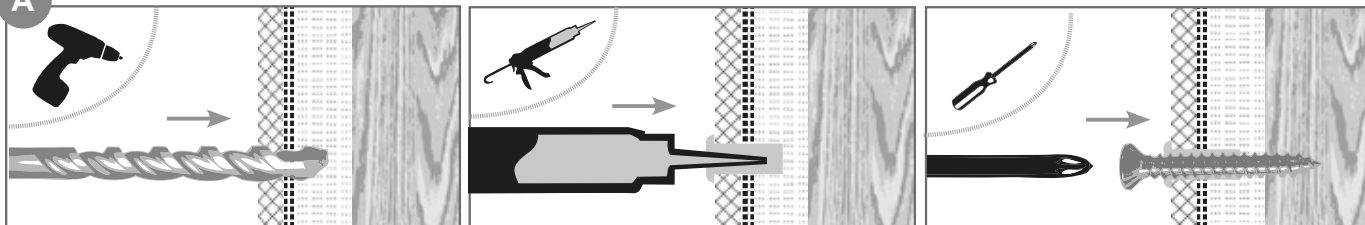
INNFESTING I MURVEGG

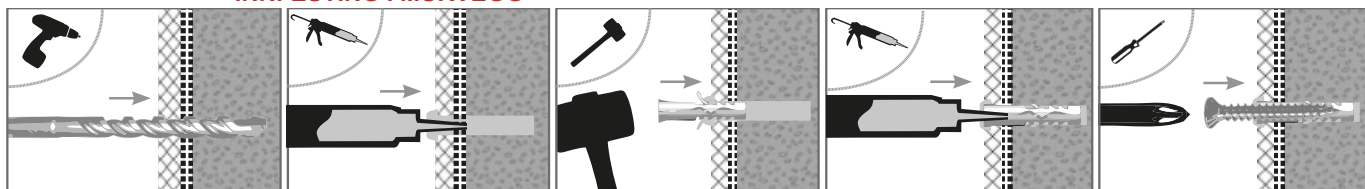Gips
 Betong
 Treverk
 Fliser

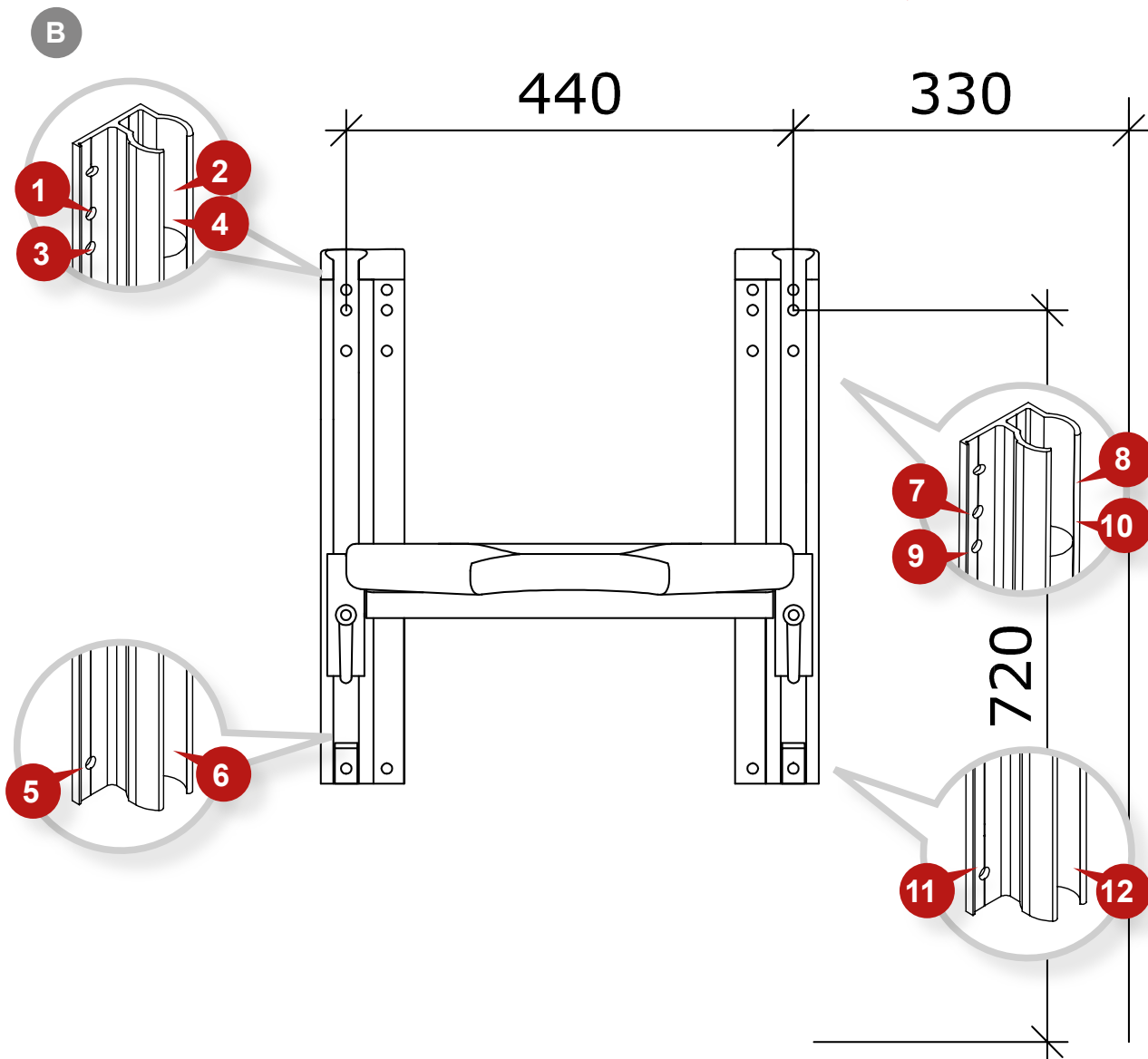ENG/ BANO SHOWER SEAT with height adjustment

5700-02 Shower seat with height adjustment, grey

5700-B-02 Shower seat with height adjustment. black

CONTENTS:

1 height-adjustable shower seat with 2 wall brackets	1 left aluminium bracket cover
1 right aluminium bracket cover	2 aluminium stoppers
1 right plastic end cap	12 acid resistant m8x60 screws and discs (delivered separately)
1 left plastic end cap	

PLACEMENT ON WALL

The recommended distance between the two brackets (between the screw holes 1/3 and the screw holes 8/10) is 440 millimeters. Recommended minimum distance from side wall to the screw holes 1/8 is 330 millimeters. The recommended height from floor to screw holes 1/8 is 720 millimeters. See figure **B**

Measure distances, level the brackets vertical and mark the screw holes No.1-12 as shown in the figure.

Please note that the two top holes in the bracket is not to be used for fixing bracket to the wall.

FIXING TO WALL

Make sure never to perforate wet walls completely. Clean the holes after drilling. Make sure to use sealant approved for application on wet walls. See figure **A**

Please note that the product guarantee only applies when the wall and screws are dimensioned according to «Byggebransjens Våtromsnorm» and Bano interface for products mounted in Norway. For products mounted in other countries, please note that the product guarantee only applies when wall and screws are dimensioned according to national guidelines/regulations and Bano interface.

Tiled wooden wall:

Make sure never to perforate wet walls completely. Drill holes with a 10 mm drill bit through the tiles only. Clean and fill the hole with sealant approved for application on wet walls. See illustration **A**

Tiled concrete wall:

Drill holes using a 12 mm drill bit through the tiles only. Switch to a 10 mm drill bit for further drilling through the concrete. Fill the hole with sealant, place a nylon plug in the hole and fill the plug with sealant. See illustration **A**

Other wall materials (bathroom boards, vinyl):

Make sure never to perforate wet walls completely. Drill holes which are dimensioned for said type of wall. Fill the hole with sealant, place a nylon plug in the hole and fill the plug with sealant. See illustration **A**

INSTALLATION

Mount the brackets to the wall with two screws at the lower end and four screws at the upper end of each wall bracket as shown in figure **B**. Fasten stopper in screw hole 6 and 12 with angle pointing up.

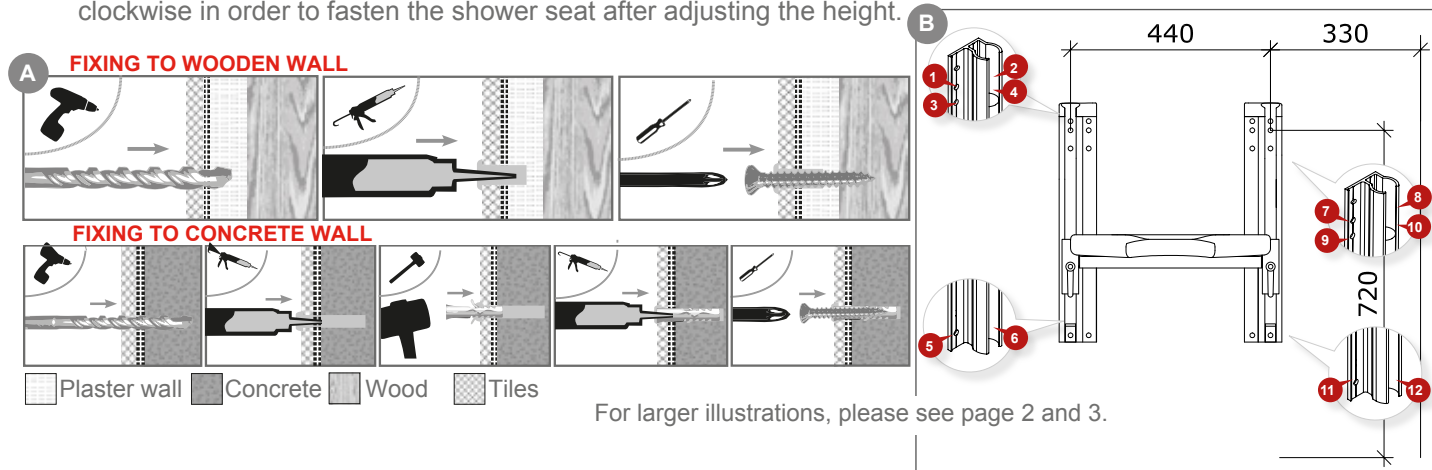
Make sure to use acid resistant screws.

COVER INSTALLATION

Slide the bracket covers into the wall brackets from above with the bent ends pointing upwards. Push the end caps in place at the top of the wall brackets so that the rounded barbs of the end caps click into place in the two upper screw holes, closest to the upper edge of the brackets.

HEIGHT ADJUSTMENT

Push the safety handle and turn it counter-clockwise to loosen the shower seat. Hold on to the shower seat to make sure that it does not slide down to the top of the brackets. Push and turn the safety handle clockwise in order to fasten the shower seat after adjusting the height.

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